OrthoPrax Air Walker Pro

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax Air Walker Pro is a medical device. It is an orthosis for the lower leg and foot to provide immobilization in a specified position.

Indications

Tall

- Conservative / post-operative care for the foot & ankle
- Immobilization of the lower leg / foot region
- Ankle sprains (severe)
- Stable fractures
- Soft tissue injuries

Short

- Conservative / post-operative care for the foot & ankle
- Immobilization of the forefoot and midfoot region
- Ankle sprains (moderate to severe)
- Metatarsal fractures
- Soft tissue injuries

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- Once you have been prescribed the OrthoPrax Air Walker Pro, only use it according to your indication and any additional instructions given by a medical specialist². Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

- Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:
- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
 - Varicose veins (varicosis)
 - Impaired sensation in the leg due to neurological abnormalities
 - Circulatory disorders of the leg (e.g. in cases of diabetes mellitus)
 - Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Putting on and taking off the OrthoPrax Air Walker Pro

Deflate before putting on the product by pressing the valves next to the bulbs.

1. Undo the straps, remove the front shell, and open the liner.
2. While seated, place the leg into the orthosis, ensuring the heel is all the way to the back of it. Surround the leg and foot with the liner and secure it.

3. Place the front shell on the back of the foot and fix the shell in place with the straps.
4. Pull the straps through the loops from the bottom up and fasten them.
5. Fasten the Velcro straps starting at the toes.
6. Inflate the liner in order to reach the desired stability level.
7. The toe cap can be attached or removed to protect from impact.
8. Deflate before taking off the product by pressing the valves next to the bulbs.

Cleaning instructions

Remove the inner cushion (liner) of the walker before washing it. Please use mild detergent and a mesh bag.

To clean the plastic parts, use a moist cloth. Never expose your product to direct heat / cold. Also observe the washing symbols in the instructions for use. Regular care guarantees optimum effectiveness of the product.

General information for specialists

Instruct the patient how to put on the orthosis correctly, and in particular how to use the pump and valve. The air chambers can be operated independently of one another using the 2 bulbs. Check that the inner cushion is positioned correctly and without creases. To prevent pressure points from forming, use the additional enclosed cushion.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax Air Walker Pro have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polypropylene (PP), thermoplastic rubber, nylon, TPU-65A, nylon fabric, TPU, PU foam, EVA

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**Span Link International LLC**