OrthoPrax Ankle Brace 1902

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax Ankle Brace 1902 is a medical device. It is an ankle orthosis for stabilization with limitation of pronation and supination.

Indications

- Conservative treatment of ankle sprains
- Chronic ligament instability
- Post-operative care
- Secondary prevention in cases of ankle ligament injuries
- Ligamentous hyperlaxity

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- Once you have been prescribed the OrthoPrax Ankle Brace 1902, only use it according to your indication and any additional instructions given by a medical specialist². Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

- Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:
- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
 - Varicose veins (varicosis)
 - Impaired sensation in the leg due to neurological abnormalities
 - Circulatory disorders of the leg (e.g. in cases of diabetes mellitus)
 - Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Putting on and taking off the OrthoPrax Ankle Brace 1902

1. Undo the laces by opening the two Velcro tabs and open the orthosis wide so you can put it on easily.
2. Put on the orthosis. The laces face forward and the heel is positioned in the heel opening. Make sure that there are no creases in the tongue.
3. Pull both laces with the Velcro tabs at the same time in the direction of the heel and fasten them there.

4. Pull the ankle stabilization straps one after another over the back of the foot and fasten them under the foot (in front of the heel) on the opposite side. The straps will cross at the back of the foot.
5. Wrap the upper elastic strap in a circle around the lower leg using light tension and fasten it in place.
6. To take off the brace, open the straps wide in reverse order to protect the ankle while taking it off.

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Regular care guarantees optimum effectiveness of the product.

General information for specialists

Instruct the patient how to put on the orthosis correctly. The lateral stabilizing stays are removable.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax Ankle Brace 1902 have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyester (PES); nylon; Polypropylene (PP)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**Span Link International LLC**