OrthoPrax Ankle Stirrup

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax Ankle Stirrup is a medical device.
It is an ankle orthosis for stabilization in the frontal plane.

Indications

- Post-operative protection after ligament suture repair / reconstruction
- Pronation and / or supination trauma
- Conservative treatment of ankle sprains (all levels of severity)
- Functional treatment of ligament ruptures
- Weber A fracture
- Chronic ligament instability

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- Once you have been prescribed the OrthoPrax Ankle Stirrup, only use it according to your indication and any additional instructions given by a medical specialist². Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins (varicosis)
- Impaired sensation in the leg due to neurological abnormalities
- Circulatory disorders of the leg (e.g. in cases of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Putting on and taking off the OrthoPrax Ankle Stirrup

1. Place your foot on the heel strap and position the two shells so that the gel cushions point towards the ankle.
2. When putting on the lateral shells, make sure they are parallel to one another and the ankle protrusions are above the ankle joint.
3. Pull the straps tight in opposite directions around the orthosis and the leg.
4. Start with the lower strap and bring it around from the front.

TIP: Cool down the orthosis in the refrigerator (not in the freezer) beforehand.

Cleaning instructions

To clean the plastic parts, use a moist cloth. Never expose your product to direct heat / cold. Regular care guarantees optimum effectiveness of the product.

General information for specialists

Instruct the patient how to put on the orthosis correctly. Adjust the heel width using the two lateral straps with Velcro fastening so that the lateral shells fit optimally. Shorten them if required.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax Ankle Stirrup have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polypropylene (PP), gel, PVC, nylon, metal (rivet)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.