OrthoPrax Knee Brace 1832

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax Knee Brace 1832 is a medical device. It is an orthosis for the increased stabilization of the knee joint using joint splints on the sides and restricting the range of motion.

Indications

- Moderate knee instability
- Cruciate and collateral ligament injury
- Meniscus injuries
- Osteoarthritis of the knee joint
- Deformities and injuries of the knee
- Post-operative treatment following knee surgery
- Knee sprains

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- Once you have been prescribed the OrthoPrax Knee Brace 1832, only use it according to your indication and any additional instructions given by a medical specialist². Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins (varicosis)
- Impaired sensation in the leg due to neurological abnormalities
- Circulatory disorders of the leg (e.g. in cases of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Putting on and taking off the OrthoPrax Knee Brace 1832

1. Open the straps and fasteners and place the orthosis around the leg, aligning the middle of the hinges on both sides of the knee.
2. Wrap the orthosis around the lower leg so that the opening is in the front and the kneecap is enclosed in the middle.
3. Wrap the upper part of the orthosis around the thigh and fasten with the Velcro fastening.

4. Tighten both straps firmly to prevent the orthosis from slipping, starting with the lower strap.

Cleaning instructions

Please use mild detergent and a mesh bag. Remove the hinge before washing. Never expose your product to direct heat / cold. Also observe the washing symbols in the instructions for use. Regular care guarantees optimum effectiveness of the product.

General information for specialists

Adjust the straps so that the hinges are positioned centrally at the knee. Instruct the patient how to put on the orthosis correctly. In particular, instruct the patient only to open the front strap.

Joint restriction:

1. Open the hinge cover to the front.
2. Insert the pins in the desired extension or flexion position.
3. Make the adjustment on both sides in the same way.
4. Close the hinge cover by snapping it in.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax Knee Brace 1832 have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyester (PES), Aluminium (Al), nylon, Polypropylene (PP)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**Span Link International LLC**