
OrthoPrax Knee Post-OP Pro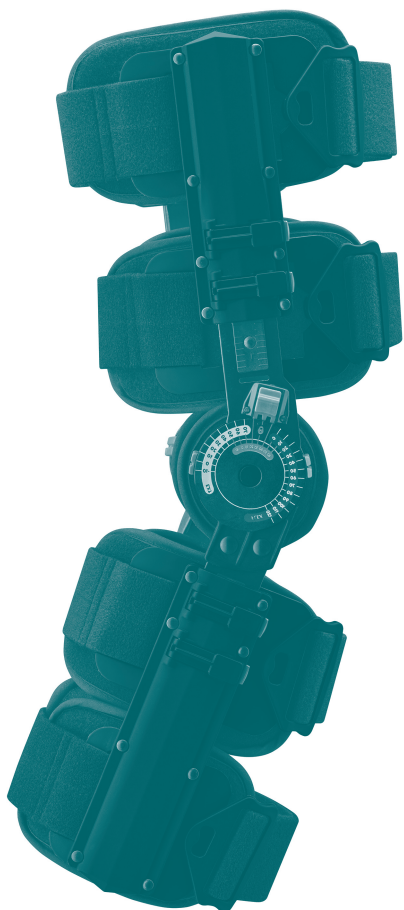

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax Knee Post-OP Pro is a medical device. It is a knee orthosis for early functional mobilization with defined ranges of motion.

Indications

- Pre-operative and post-operative care
- Post-traumatic care
- Fractures
- Ligament injuries
- Meniscus injuries

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- Once you have been prescribed the OrthoPrax Knee Post-OP Pro, only use it according to your indication and any additional instructions given by a medical specialist*. Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins (varicosis)
- Impaired sensation in the leg due to neurological abnormalities
- Circulatory disorders of the leg (e.g. in cases of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Putting on and taking off the OrthoPrax Knee Post-OP Pro

1. Open the orthosis (undo straps and buckles).
2. While sitting or lying down, position the orthosis beneath the outstretched leg and align the middle of the hinges on the sides of the knee joint.
3. Make sure that the hinge and the hinge leg are aligned with the midline of the leg, fasten both straps (below and above the knee joint) using the buckles.
4. Fasten the two remaining upper and lower straps.
5. Tighten the straps through the buckles to the desired level of tension.
6. To take off the orthosis, only use the quick-fit fasteners.

Cleaning instructions

Remove the inner cushion and straps to wash them. Please use mild detergent and a mesh bag. To clean the plastic and metal parts, use a moist cloth. Never expose your product to direct heat / cold. Also observe the washing symbols in the instructions for use. Regular care guarantees optimum effectiveness of the product.

General information for specialists

Instruct the patient how to put on the orthosis correctly.

Adjusting the orthosis:

1. To adjust the length of the orthosis, open the lateral fastener and pull the thigh and lower leg brace into the desired position. A scale helps to adjust both splints to the same length.
2. By closing the side fastener, you can fix the hinge leg in place.
3. Adjusting the joint splints to the patient's leg. Adjusting the ROM joint:
 1. The restriction of movement (range of motion, ROM) can be adjusted from -10 to 70 degrees of extension and from 0 to 120 degrees of flexion in increments of 10 degrees for either direction.
 2. Pull out the fixing mechanism of the ROM joint and push this into the desired position. Immobilization from -10 to 60 degrees can be achieved using the middle fastener.
 3. Make sure that the two joints are in the same position and setting.
 4. Check all fixing mechanisms of the splints.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax Knee Post-OP Pro have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polypropylene (PP), Aluminum (Al), nylon, PU foam

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**Span Link International LLC**