
OrthoPrax LSO 627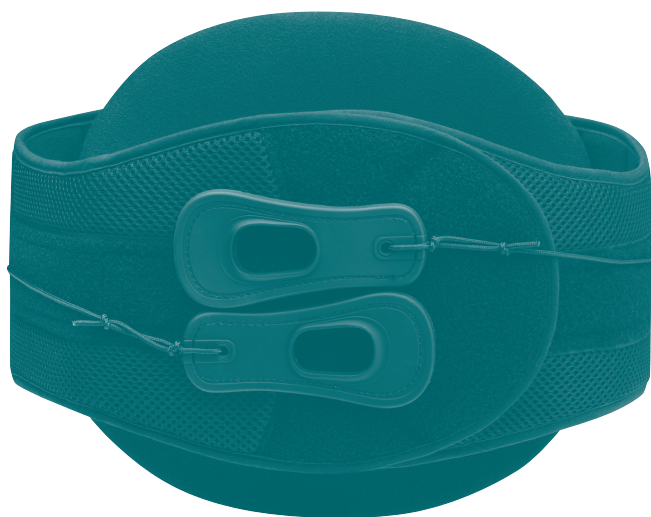

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax LSO 627 is a medical device. It is an orthosis that provides functional mobilization of the lumbar spine.

Indications

- Deformities and injuries of the lumbar spine
- Sprains
- Excessive strain on the spine
- Spondylolisthesis with lumbosciatica
- Post-traumatic treatment of vertebral fractures
- Spinal stenosis with paresis
- Lumbosciatica

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- Once you have been prescribed the OrthoPrax LSO 627, only use it according to your indication and any additional instructions given by a medical specialist². Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

- No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:
- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
 - Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion)
 - Impaired lymphatic drainage and soft tissue swelling of unknown origin
 - Pacemaker (follow physician's instructions)
 - Stoma (follow physician's instructions)
 - Chronic inflammation of organs in the abdominal area

Application instructions

Putting on and taking off the OrthoPrax LSO 627

1. While standing upright, position the orthosis so that the back section is directly centered above the waist on your spine. For orientation, the bottom edge of the orthosis can be positioned on a level with the base of the buttocks.
2. Keeping the rear stabilizing element centered on your back, pull the sides of the support around your abdomen.

3. Tighten the front fastening and close it over the abdomen. Make sure that the rear stabilizing element is centered on your spine. Adjust it as required.
4. Grip the compression straps using the thumb loops and pull them away from your body until you have created the desired compression.
5. As required, this compression level can be adjusted over the course of the day.

Cleaning instructions

Please use mild detergent. Never expose your product to direct heat / cold. Also observe the washing symbols in the instructions for use. Regular care guarantees optimum effectiveness of the product.

General information for specialists

Instruct the patient how to put on the orthosis correctly.

Adjusting the size:

1. Determine the size by measuring the waist and comparing with the size chart.
2. On the inside of the pelvic support, the corresponding sizes are marked and labeled. Fasten the strap at the correct marking and cut off the excess material using scissors.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax LSO 627 have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyester (PES), PU foam, ABS, nylon, XPE foam

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**Span Link International LLC**