OrthoPrax LSO 631

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax LSO 631 is a medical device.
It is an orthosis that provides relief and functional mobilization of the lumbar spine by reducing lordosis.

Indications

- Severe lumbar deformity in case of facet syndrome / osteoarthritis
- Post-traumatic treatment of non-severe vertebral fractures
- Sprains
- Excessive strain on the spine caused by abdominal compression
- Spondylolisthesis, grade II, with recurring lumbosciatica
- Collapsed superior end plates in the lumbar spine
- Spinal canal decompression

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- OrthoPrax LSO 631 unload must only be worn in accordance with the specifications contained in these instructions and used for the indications listed (part of the body this product is used for). To guarantee the best possible fit, it is essential to check whether the product is adequately shaped and whether it needs to be anatomically reshaped by a specialist. Only a specialist may carry out the (initial) fitting of the product and provide instructions.
- The OrthoPrax LSO 631 must be applied over a T-shirt or undershirt.
- The use in combination with other products, e.g. as part of compression treatment (compression pantyhose), must be discussed beforehand with your physician.
- If you notice any unusual changes or your symptoms get worse, please contact your physician without delay.
- No product liability is accepted in the event of improper use.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.
- Side effects involving the body as a whole have not been reported to date. Correct usage / fitting is assumed. Do not fit the OrthoPrax LSO 631 too tightly, as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. In this case, loosen the straps and, if necessary, check the size of your product.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin
- Pacemaker (follow physician's instructions)
- Ostomy (follow physician's instructions)
- Chronic inflammation of organs in the abdominal area

Application instructions

Putting on and taking off the OrthoPrax LSO 631

1. While standing upright, position the orthosis so that the back section is directly centered above the waist on your spine. For orientation, the bottom edge of the orthosis can be positioned on a level with the base of the buttocks.
2. Keeping the rear stabilizing element centered on your back, pull the sides of the support around your abdomen.
3. Tighten the front fastening and close it over the abdomen. Make sure that the rear stabilizing element is centered on your spine. Adjust it as required.
4. Insert your thumbs into the compression strap loops and pull them away from your body until you have created the desired compression. Then attach the loops on the abdominal fastening.
5. As required, this compression level can be adjusted over the course of the day.

Reinigungshinweise

Wash the product by hand. Never expose your product to direct heat / cold. Also observe the washing symbols in the instructions for use.

General information for specialists

Instruct the patient how to put on the orthosis correctly.

Adjusting the size:

1. Determine the size by measuring the waist and comparing with the size chart.
2. On the inside of the pelvic support, the corresponding sizes are marked and labeled. Fasten the strap at the correct marking and cut off the excess material using scissors.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax LSO 631 have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyester (PES), PU foam, Polypropylene (PP), nylon, XPE foam

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.