OrthoPrax Shoulder Immobilizer 15

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax Shoulder Immobilizer 15 is a medical device. It is a shoulder orthosis that provides relief and immobilization in a 15° abduction position.

Indications

- Pre-operative and post-operative immobilization
- Acute irritation
- Post-traumatic, conservative care
- Soft tissue injuries in the shoulder region
- Treatment of chronic pain

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- Once you have been prescribed the OrthoPrax Shoulder Immobilizer 15, only use it according to your indication and any additional instructions given by a medical specialist². Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

- Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:
- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
 - Varicose veins (varicosis)
 - Impaired sensation in the arm due to neurological abnormalities
 - Circulatory disorders of the arm (e.g. in cases of diabetes mellitus)
 - Impaired lymphatic drainage and soft tissue swelling of unknown origin
 - Pacemaker (follow physician's instructions)
 - Amputation of the breast (follow physician's instructions)

Application instructions

Putting on and taking off the

OrthoPrax Shoulder Immobilizer 15

1. Guide the arm into the pocket and take hold of the handle.
2. Place the cushion against the body with the arched side facing forward.
3. The shoulder strap is guided over the head. If required, the quick-fit fastener can be opened to put it on.

4. The waist strap is guided around the body and closed with the quick-fit fastener.
5. To take off the support, proceed in the opposite order.

Cleaning instructions

Before washing, remove the top cover of the shoulder cushion. Please use mild detergent and a mesh bag. Never expose your product to direct heat / cold. Regular care guarantees optimum effectiveness of the product.

General information for specialists

Instruct the patient how to put on the orthosis correctly.

1. The length of the arm pocket and the position of the handle can be adjusted for a correct fit by folding back and adjusting the Velcro fastenings. Position the handle and fix it in place by adjusting the three Velcro fastenings at the top and bottom of the handle.
2. Position the arm in the pocket and the cushion on the waste and adjust the length of the straps.
3. Place the shoulder strap over the uninjured shoulder and fasten it to the buckle on the handle (sling).

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax Shoulder Immobilizer 15 have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyester (PES); PU foam;
Polyoxymethylene (POM); nylon

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**Span Link International LLC**