OrthoPrax Shoulder Post-OP Pro

Dear customer,

please read and observe these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended use

The OrthoPrax Shoulder Post-OP Pro is a medical device. It is a shoulder orthosis that provides relief and immobilization at adjustable angles.

Indications

- Pre-operative and post-operative care
- Condition after reconstruction of tendons or ligaments in the shoulder (capsular ligaments)

Risks of using this product

- Once you have been prescribed the OrthoPrax Shoulder Post-OP Pro, only use it according to your indication and any additional instructions given by a medical specialist*. Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

- Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:
- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
 - Varicose veins (varicosis)
 - Impaired sensation in the arm due to neurological abnormalities
 - Circulatory disorders of the arm (e.g. in cases of diabetes mellitus)
 - Impaired lymphatic drainage and soft tissue swelling of unknown origin
 - Pacemaker (follow physician's instructions)
 - Amputation of the breast (follow physician's instructions)

Application instructions

Putting on and taking off the OrthoPrax Shoulder Post-OP Pro

1. Guide the arm into the pocket and take hold of the handle.
2. The shoulder strap is guided over the head. If required, the quick-fit fastener can be opened to put it on.
3. Place the rigid waist support against the body, above the hips, with the arched side facing forward.
4. The waist strap is guided around the body and closed with the quick-fit fastener.
5. To take off the support, proceed in the opposite order.

Cleaning instructions

Please use mild detergent. Never expose your product to direct heat / cold. Also observe the washing symbols in the instructions for use. Regular care guarantees optimum effectiveness of the product.

General information for specialists

Instruct the patient how to put on the orthosis correctly.

Adjusting the orthosis:

1. Shape the waist support to anatomically fit the patient on the side requiring support.
2. To choose the rotation angle, bend the long metal support into the desired position. Shape the short metal piece against this to secure the rotation angle.
3. Optional: To adjust an abduction angle, you will need the enclosed 90 degree bracket and 4 screws and nuts.
 - Loosen all nuts and screws on the metal supports for the rotation angle and remove them.
 - Screw the 90 degree bracket in place behind the waist support into the 4 holes using the same nuts and screws and bend to the desired level of abduction.
 - Screw the previously loosened metal supports onto the elongated slots by hand.
 - During fitting with the patient, slide this along the elongated slots into the correct position and tighten the screws.

Adjusting the arm sling:

1. The length of the arm pocket and the position of the handle can be adjusted for a correct fit by folding back and adjusting the Velcro fastenings. Position the handle and fix it in place by adjusting the bottom tensioning strap.
2. Position the arm in the pocket and the cushion on the waste and adjust the length of the straps.
3. Place the shoulder strap over the uninjured shoulder and fasten it to the buckle on the handle (sling).

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax Shoulder Post-OP Pro have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Nylon; Polyester (PES); Aluminum (Al); Polyethylene (PE); PU foam

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**Span Link International LLC**