OrthoPrax Wrist Brace 3807

Dear customer,

please read through these instructions for use carefully.
If you have any questions, please contact your doctor or medical retailer.

EN

Intended purpose

The OrthoPrax Wrist Brace 3807 is a medical device. It is an orthosis for immobilizing the wrist, combined with a thumb immobilizer.

Indications

- Acute, post-traumatic and post-operative irritation and injuries of the wrist, carpal area and thumb ray
- Pre-operative care
- Tenosynovitis
- Osteoarthritis of the first carpometacarpal joint
- Immobilization following nerve lesions
- Carpal tunnel syndrome

Risks of using this product

- Please closely observe the specifications in these instructions for use and the instructions given by the specialist.
- Once you have been prescribed the OrthoPrax Wrist Brace 3807, only use it according to your indication and any additional instructions given by a medical specialist². Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins (varicosis)
- Impaired sensation in the arm due to neurological abnormalities
- Circulatory disorders of the arm (e.g. in cases of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Putting on and taking off the OrthoPrax Wrist Brace 3807

1. Unfasten the Velcro and open the orthosis wide.
2. Insert your hand into the orthosis up until the thumb strap.
3. Using the laces, tighten the strap and fasten the Velcro on the orthosis.
4. If needed, retighten the thumb strap.

Cleaning instructions

Wash the product by hand. Never expose your product to direct heat / cold. Also observe the washing symbols in the instructions for use.

General information for specialists

Before putting on the orthosis, ensure the correct fit of the metal splints. Resize if needed. Verify the correct angle of the thumb joint, which you can adjust using the inner splint. Instruct the patient how to put on the orthosis correctly.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OrthoPrax Wrist Brace 3807 have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Nylon; Polyester (PES); Aluminium (Al); PU foam

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

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Bauerfeind USA, Inc.
**Span Link International LLC**